








THUASNE

Lombamum®

fr	Ceinture lombo-sacrée de maternité	6
en	Lumbo-sacral belt for maternity	6
de	Lumbosakral-Stützgürtel für Schwangere	7
nl	Lumbosacrale zwangerschapsbrace	8
it	Cintura lombo-sacrale per gravidanza	9
es	Faja lumbosacral de maternidad	10
pt	Cinta lombossacral para grávidas	11
da	Lumbosakralt graviditetsbælte	12
fi	Lannerangan ja ristiselän tukivyö raskausajalle	13
sv	Lumbosakralt stödbälte graviditet	14
el	Zύνη οσφύος μπρόπτας	14
cs	Těhotenský bederní pás	15
pl	Orteza lędźwiowo-krzyżowa ciążowa	16
lv	Grūtniečiabas jostasvietas un krustu dājas josta	17
lt	Juosmens ir kryžkaulio diržas nėščiosioms	18
et	Rasedusaegne nimme-ristluu vöö	19
sl	Nosečniški oporni križno-ledveni pas	20
sk	Tehotenský lumbo-sakrálny pás	20
hu	Terhességi ágyék-keresztcsonti fűző	21
bg	Лумбо-сакрален колан за бременно	22
ro	Centură lumbosacrală pentru gravide	23
ru	Пояснично-крестцовый корсет для беременных	24
hr	Lumbalno-sakralni potporni pojasa tijekom trudnoće	25
zh	产妇腰部支撑带	26
ar	حزام للصدر وأسفل الظهر للولادة	27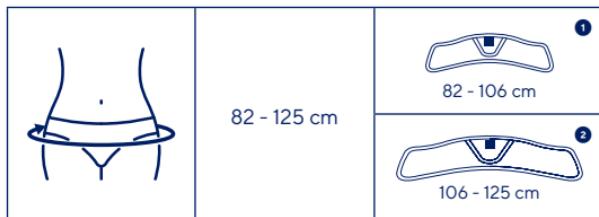

fr	Tour de hanches	Hauteur : 21 cm
en	Hip measurement	Height: 21 cm
de	Hüftumfang	Höhe: 21 cm
nl	Heupomtrek	Hoogte: 21 cm
it	Circonferenza fianchi	Altezza: 21 cm
es	Circunferencia de cadera	Altura: 21 cm
pt	Perímetro de anca	Altura: 21 cm
da	Hoftemål	Højde: 21 cm
fi	Lantionympärys	Korkeus: 21 cm
sv	Höftvidd	Höjd: 21 cm
el	Περιφέρεια γοφών	Ύψος: 21 εκ.
cs	Obvod boků	Výška: 21 cm
pl	Obwód w biodrach	Wysokość: 21 cm
lv	Gurnu apkārtmērs	Augstums: 21 cm
lt	Klubų apimtis	Aukštis: 21 cm
et	Puusaümbermõõt	Kõrgus: 21 cm
sl	Obseg bokov	Višina: 21 cm
sk	Obvod panvy	Výška: 21 cm
hu	Csípőbőség	Magasság: 21 cm
bg	Обиколка на ханша	Височина: 21 см
ro	Circumferință soldurilor	Înălțime: 21 cm
ru	Окружность бедер	Высота: 21 см
hr	Obujam bokova	Visina: 21 cm
zh	臀围	高度 : 21 cm
ar	حياط الوركين	الارتفاع: 21 سم

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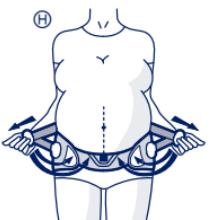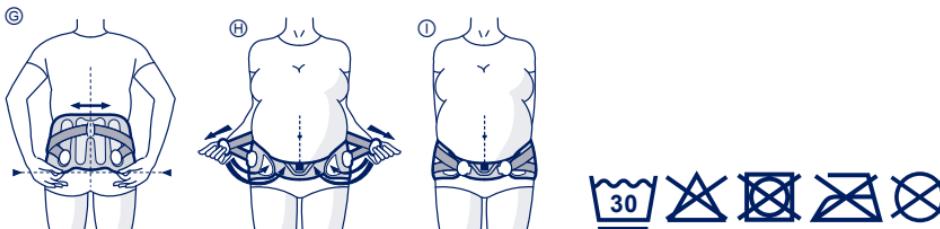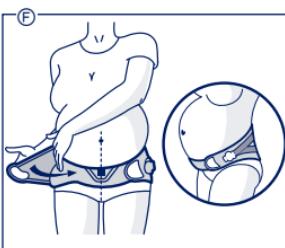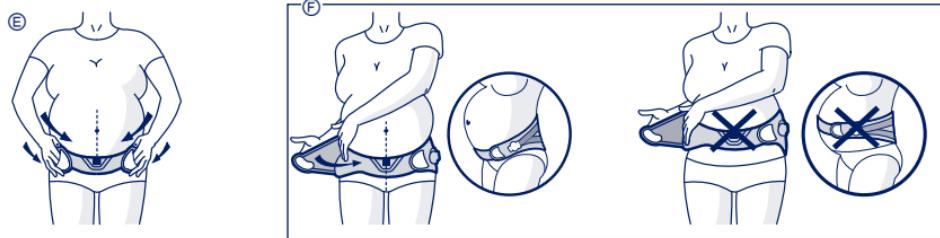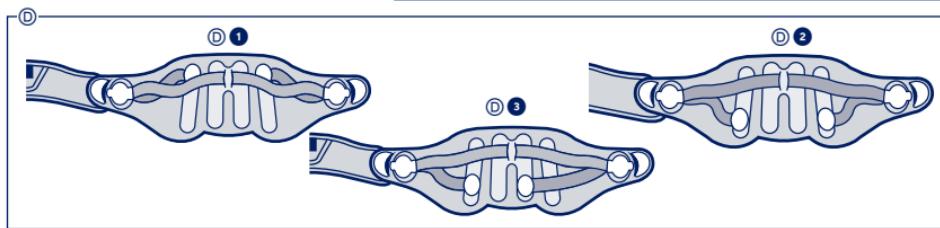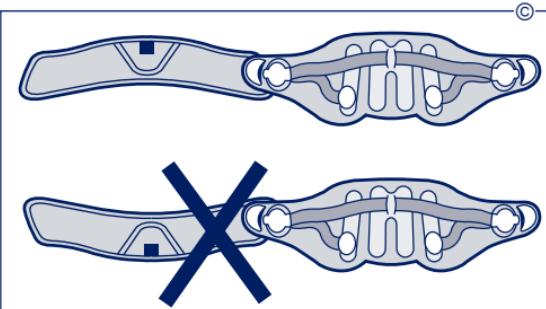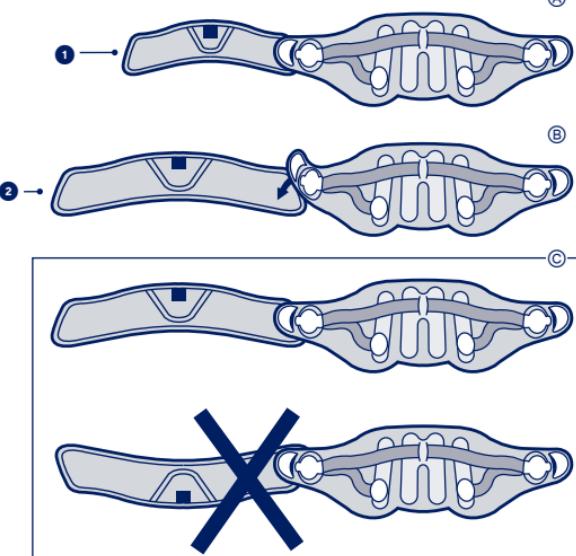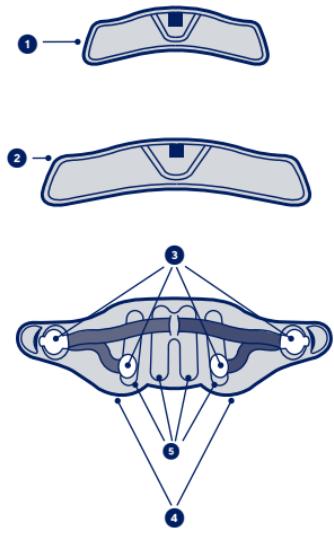LUMBO-SACRAL BELT FOR MATERNITY

Description/Destination

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

Composition

Comfort bamboo fibre lining.

Two abdominal support bands of different sizes (one band corresponding to a hip measurement of < 106 cm ①, the second corresponding to a hip measurement of > 106 cm ②). Adjustable, doubled additional strap with discs. ③

Centering tabs. ④

Conformable dorsal stays. ⑤

Texile components: polyamide - polyurethane - polyester - elastane - elastodiene - viscose

Non-textile components: brushed tempered steel - poloxymethylene

Properties/Mode of action

The device relieves the back from the baby's weight.

The device ensures an adapted abdominal support: the abdominal support bands closely fits the body throughout the pregnancy and provides firm support for the abdomen.

Indications

Pregnancy-related low back pain.

Pregnancy-related pain girdle pain (posterior pelvic pain, sacro-iliac pain, symphyseal pain).

Contraindications

Do not use in the event of known allergy to any of the components.

Do not apply the product in direct contact with broken skin.

Precautions

Verify the product's integrity before every use.

Wear the device above thin clothing.

Do not use the device if it is damaged.

Strictly comply with your healthcare professional's prescription and recommendations for use.

It is recommended that a healthcare professional adjust the stays to the shape of the patient's back before the belt is worn for the first time.

Do not wear the product in a medical imaging machine.

Do not wear the product while sleeping.

In the event of discomfort, significant hindrance, pain, abnormal sensations, remove the device and consult a healthcare professional.

It is recommended to adequately tighten the device to achieve support without excessive compression.

Undesirable side-effect

This device can cause skin reactions (redness, itching, irritations...).

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

Fitting the device

The abdominal support band already in place on the belt is the band corresponding to a hip measurement of < 106 cm. ⑥

If it is the right size for you, go straight to the fitting instructions. If your hip measurement is greater than 106 cm: remove the abdominal band in place on the belt. Replace it with the other one then attach it to the dorsal part of the belt using one of the two self-fastening parts. ⑦

The two labels must face the same way.

The point of the "V" on the middle of the band must be pointing downwards. ⑧

Adjustment of the additional strap ⑨

It is also possible to change the position of the discs located at the back:

-if they are positioned high up, it is the top of your back that will be supported, ⑩ ⑪

-if they are positioned low down, it is the bottom of your back that will be supported, ⑫ ⑬

In the case of sciatica, it is possible to provide more active support for the painful side by adjusting the additional strap located on the same side. ⑭ ⑮

Fitting the device

Position the belt using the finger loop. ⑯

The point of the "V" must be pointing downwards.

Fasten the belt and adjust it: the "V" must be vertically in line with the navel. ⑯

The belt must be centered on the spine: the dorsal stays should be positioned on either side of the spine. ⑯

If it is not, adjust using the centering tabs located on the back. The bottom of the belt must stop at the top of the crease between the buttocks.

For an adapted support, you can use the additional strap.

Pull the additional strap simultaneously and evenly. ⑯

Attach the tabs on the centre of the abdominal band. ⑯

The level of support can be adjusted as required throughout the day.

Care

Close the self-fastening tabs before washing. Machine washable at 30°C (delicate programme). Do not machine wash the device more than 10 times. Do not dry clean. Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Do not tumble-dry. Do not iron. Squeeze out excess water. Dry away from any direct heat source (radiator, sun, etc.). Dry flat.

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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Placer bæltet ved hjælp af fingerstroppen. ©
Thuasne SAS – SIREN/RCS Nanterre 542 091 186 – capital 1 950 000 euros